

Bolster Sponges

Cleaning & Care Instructions

To ensure the longevity of your new Stealth-Cote sponges, we recommend following these guidelines for cleaning and care.

Cleaning & Care:

We all know that in a medical environment, it is easy for surfaces to become dirty and soiled from blood, bodily fluids, betadine, barium and other fluids.

Do:

- Techno-Aide recommends cleaning with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean and remove dust and dirt from your coated sponge.
- To disinfect your sponges, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.
- PDI Sani-Cloth Bleach wipes can be used as a disinfectant if you strictly follow the manufactures' guidelines. Failure to comply with these guidelines will result in void of warranty.
- Store your sponges in a clean and dry area when not in use.

Don't:

- Do not autoclave or launder.
- Do not soak/submerge sponges in water.

Failure to adhere to the above recommendations will shorten the life of your bolster sponges, promote discoloration of the surface, and will possibly void your warranty.