





**ImageWorks**  
Generations of Imaging

# Intraskan DC

## REMOTE KEYPAD

**NOTE:** Set Jumpers on external console PCB to J1= J2 = Closed, J3=Open.  
Use RJ45-8P8C Cable Max. length 35 feet

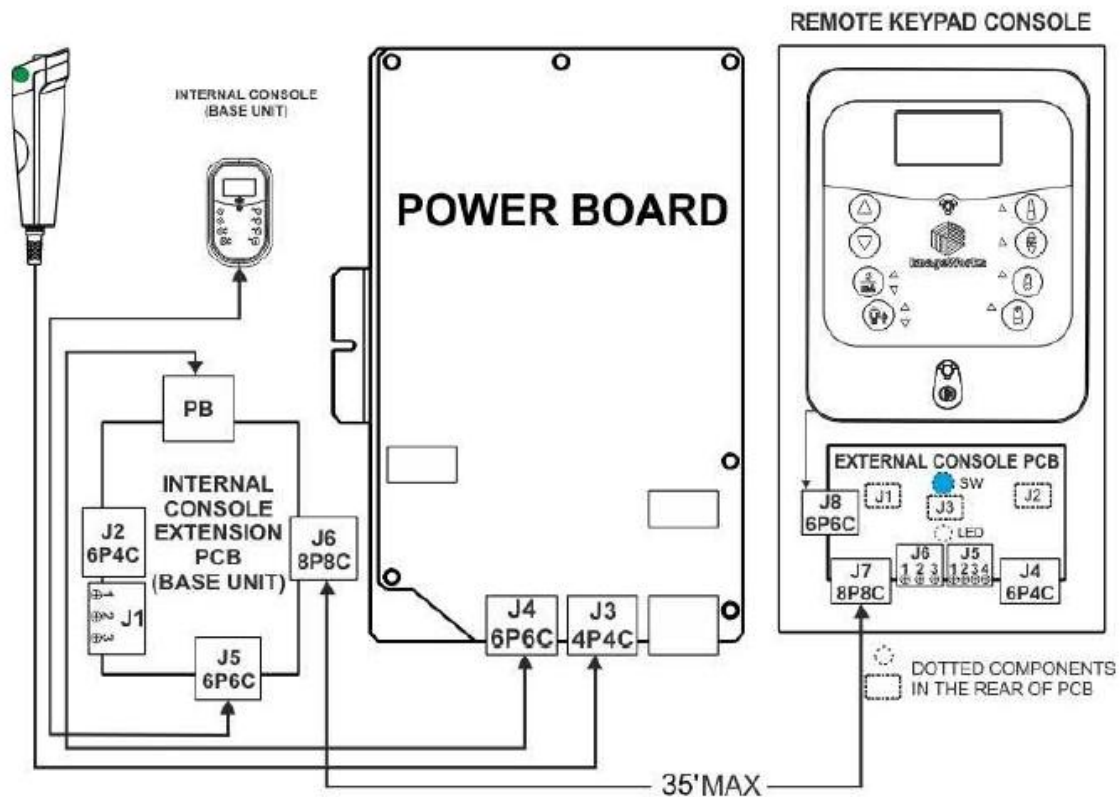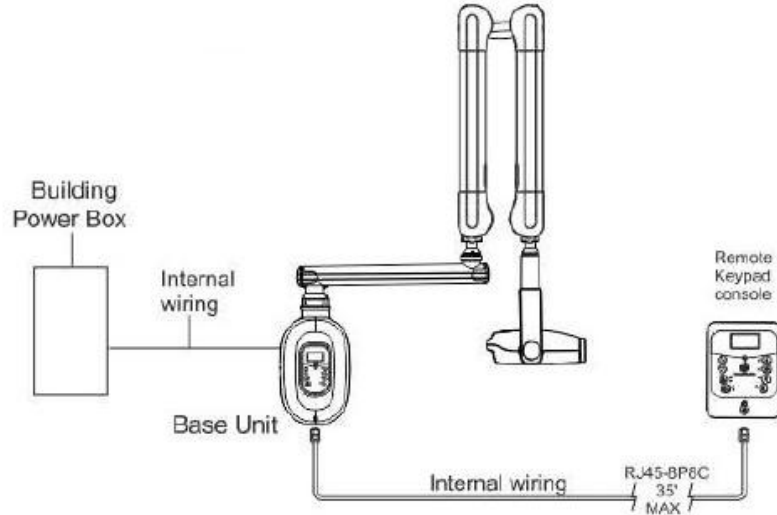



**ImageWorks**  
Generations of Imaging

# Intraskan DC

## DOORBELL SWITCH

**NOTE:** Set Jumpers on external console PCB to J1= J2 = J3 = Open.  
Use 3 conductor low voltage wire max. length 35 feet.

