


**Understanding the Acquire Tab**

**Endo Mode** allows you to take **multiple x-rays for one tooth position**

**Series Mode** allows you to take a **series of x-rays**

**Retake or Skip** an x-ray, or **Reset** the entire x-ray series order

Select your chart with the drop down menu

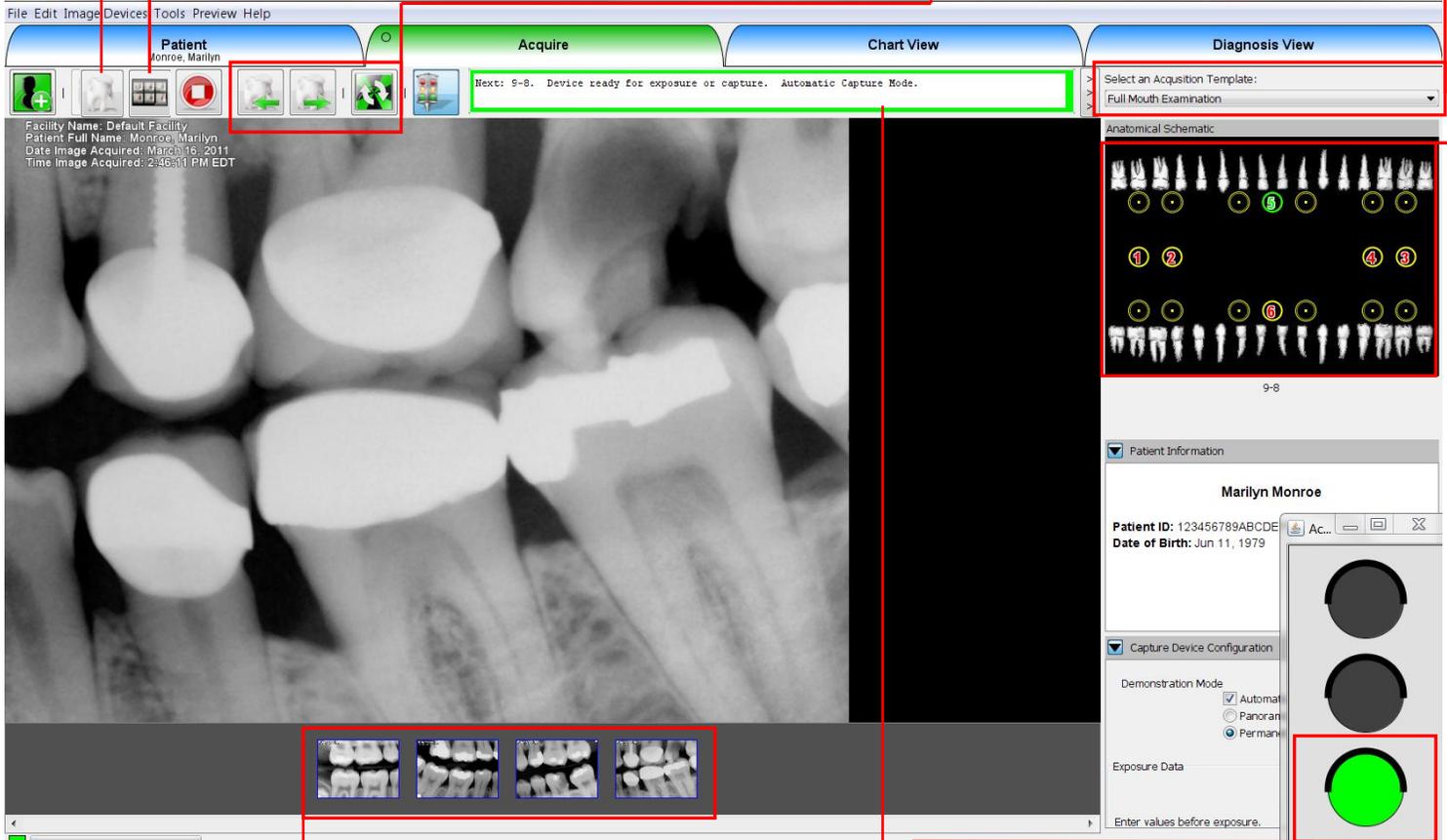

As you take x-rays they will load in order in the film strip

Click the chart positions in the order in which you would like to take your x-ray series

**Green** means go, take your next x-ray.  
**Yellow** means processing x-ray.  
**Red** means the series is complete.

**Acquire Tab Notes:**

1. You can show or hide the traffic light by clicking the button on the tool bar.
2. The active patient is always named under the Patient Tab, as well as on the right hand side.
3. In the Capture Device Configuration box in the lower right you can add exposure data to your notes on the x-ray.
4. You can select any x-rays that you would like to take, in any order. If you don't select an order the default numbering sequence will be used as is indicated in each chart.