


**Understanding the Diagnosis View Tab**

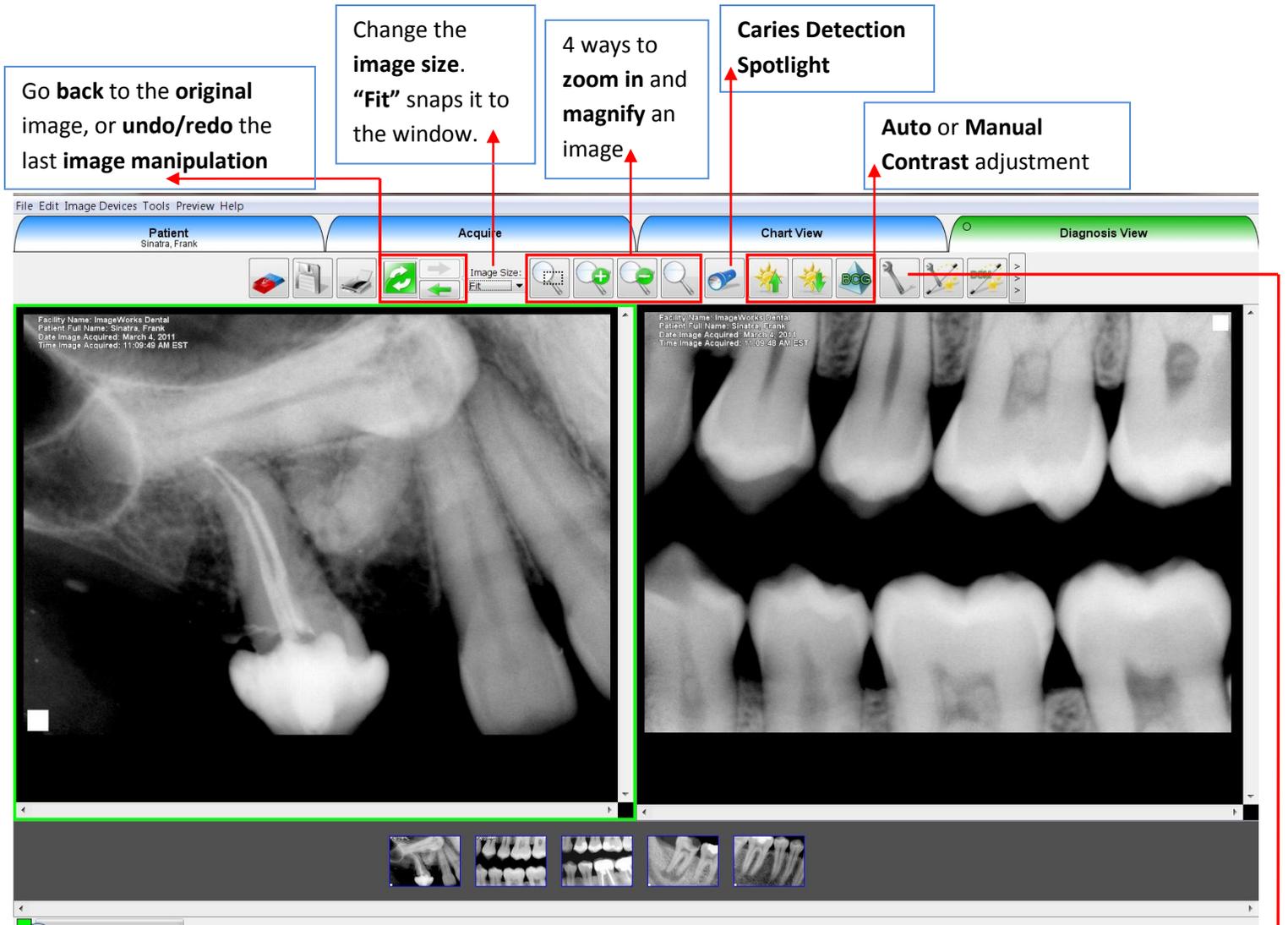

Go back to the original image, or **undo/redo** the last image manipulation

Change the **image size**. **"Fit"** snaps it to the window.

4 ways to **zoom in** and **magnify** an image

**Caries Detection Spotlight**

**Auto or Manual Contrast adjustment**

File Edit Image Devices Tools Preview Help

Patient Sinatra, Frank

Acquire Chart View Diagnosis View

Image Size: Fit

Facility Name: ImageWorks Dental  
Patient Full Name: Sinatra, Frank  
Date Image Acquired: March 4, 2011  
Time Image Acquired: 11:09:49 AM EST

Facility Name: ImageWorks Dental  
Patient Full Name: Sinatra, Frank  
Date Image Acquired: March 4, 2011  
Time Image Acquired: 11:09:49 AM EST

The **wrench icon** is the **View All Tools** icon. Click this to see all of the available **image filters** and **manipulations**, including **Sharpen**, **Measurement**, **Invert**, **Colorize**, **Draw Circles**, add **Notes** and many others.

**Diagnosis View Tab Notes:**

1. Left click the image on the right or left to make it the active image.
2. Original images are always saved as originals and cannot be overwritten. The image modifications need to be saved in order to be called back up at a later date. Any saved modified images will not overwrite nor replace the original image.
3. Use the Caries Spotlight to help you diagnosis decay and abscesses.